

# Instruction Manual



## ASSEMBLY INSTRUCTIONS



1. Open your Roller Walker. Locate the pair of folding hinges (found under the seat) and push them down until they lock securely in place.



2. Place both loose handles, with the pre-assembled locking loop brakes, into the frame and adjust to the desired height. To do this, remove the knob, washer and screw from the frame and insert the screw, with the washer on it, through the desired hole and screw the knob back on securely.



3. Insert removable backrest into Roller Walker frame securely by making sure the retainer pins are locked into position through the frame holes on both sides.



4. Before use test and adjust the tension on the braking system mechanism to ensure proper and secure braking. When Roller Walker is in use, clearance between the wheel and brake mechanism should be 3mm-5mm.

**To adjust brakes:** Use a small spanner to hold the locking nut above the wheel steady, whilst tightening or loosening the cable tensioner as required. For effective operation please ensure that both brakes are similarly adjusted.

5. Place bag under seat and secure straps around front and centre crossbars.

## SPECIFICATION

- Wheels: 8"
- Bag: 42cm(w) x 27cm(d) x 15cm(h)
- Handle height: 85cm – 96cm
- Width between handles: 45cm
- Seat: 35cm(w) x 37cm(d)
- Seat height: 58cm
- Weight capacity: 115Kg
- N.W./G.W.: 7Kg/8Kg

## PARTS LIST

- 2 Handlebars with Loop Lock Brakes
- 1 Bag
- 1 Backrest

### WARNING!

It is very important that you read all instructions. The maximum safe weight capacity is 115Kg (18 stone), please DO NOT exceed it!



Distributed by The Consortium®  
[www.homecare-products.co.uk](http://www.homecare-products.co.uk)

Because of continuing improvements, actual product may differ slightly from photo.